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Datum

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This screenshot shows the 'Redigera ansvariga vårdpersonal' dialog box overlaid on the patient list. The dialog has two dropdown menus: 'Ansvarig sjuksköterska' (with 'Harris Testson' selected) and 'Ansvarig läkare' (with 'Mila Testberg' selected). A red dashed box highlights these dropdowns, with a red circle '2' next to it. A red arrow points from a red circle '1' to the edit icon (pencil) on the patient list row. A red circle '3' is next to the 'Slutför' button at the bottom right of the dialog.

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The screenshot displays a patient monitoring dashboard for Sofia Testlöf (75 år, 19). The interface includes a patient list, a detailed view of the patient's health data, and a graph of blood pressure. Red circles and arrows indicate the following steps:

- 1**: Patient selection in the list.
- 2**: Clicking on the 'Öppna graf' button below the graph.
- 3a**: Clicking the edit icon (pencil) for a blood pressure measurement.
- 3b**: Clicking the 'Sätta' button to set a target value.
- 3c**: Clicking the 'Slutför' button to confirm the update.
- 4a**: Clicking the delete icon (trash) to remove a measurement.
- 4b**: Clicking 'Ja' in the confirmation dialog 'Ta bort mät'.

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The image shows a patient monitoring interface for Sofia Testlöf. The top section displays patient information and the latest measurement: 110/75 mmHg for 5 days ago. Below this, there are three main panels:

- Panel 1 (Left):** Shows 'Hälsodata' with a line graph of blood pressure. A red circle '2' highlights the 'Öppna graf' button.
- Panel 2 (Middle):** Shows 'Hjärtvikt' with a line graph of blood pressure. A red circle '4' highlights the 'Spara gränsvärden' button in the settings dialog.
- Panel 3 (Right):** Shows 'Egenmonitoreringsplan' with a settings dialog for 'Spara gränsvärden'. A red circle '3' highlights the 'Redigera gränsvärden' button, a red circle '5' highlights the 'Spara gränsvärden' button, and a red circle '7' highlights the 'X' close button.

The settings dialog for 'Spara gränsvärden' shows a scale for 'Systoliskt blodtryck' with markers for P1, P2, IV, P2, and P1. The current value is 140 mmHg.

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The screenshot shows a patient monitoring interface for Sofia Testlöf. The interface includes a patient list at the top, a patient info section, and a monitoring plan section. The monitoring plan section shows a graph of blood pressure and pulse data. The interface is annotated with numbered callouts (1-7) indicating the steps for editing patient information and setting up a monitoring plan.

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