



Skaraborgs sjukhus Skövde Akuten

Start up training 3-4/3 & 9/3 - 2026

Simcapture Pro : skasakuten-pro.simcapture.com

The screenshot shows the Simcapture Pro web interface. At the top, there is a navigation bar with 'simcapture' and menu items for Sessions, Scenarios, Reports, and Users. The user profile 'SimCapture Support, Laerdal' is visible on the right. The main content area is titled 'Welcome, Laerdal!' and features a 'Quick Debrief' section on the left. The central 'Start or Watch a Session' area contains three cards: 'SimRum', 'Akutrum 1', and 'Akutrum 3'. Each card has a 'START NEW SESSION' button and a 'LAST SESSION' field. Below these cards is a 'Your Activity' section showing a recent activity: 'Viewed Session, Demo: Strukturerat omhändert'. A 'DEBRIEF SESSION' button is located at the bottom right of the activity section. A small image of a Simcapture hardware device is also visible.



Set up home page

The image shows a user interface for setting up a home page. It features a grid of session cards under the heading "Start or Watch a Session". Each card has a "START NEW SESSION" button and a "LAST SESSION" section. A settings gear icon is circled in red, and an arrow points to a settings menu that is open, showing three checked items: SimRum, Akutrum 1, and Akutrum 3.

Top Row (Initial State):

- SimRum:** LAST SESSION (6 DAYS AGO) Demo: Strukturerat omhändertagande enligt ABCDE
- Akutrum 1:** LAST SESSION (16 DAYS AGO) Session - February 16, 2026 at 10:45
- Akutrum 3:** LAST SESSION None


Bottom Row (Final State):

- SimRum:** LAST SESSION (6 DAYS AGO) Demo: Strukturerat omhändertagande enligt ABCDE
- Akutrum 1:** LAST SESSION (16 DAYS AGO) Session - February 16, 2026 at 10:45
- Akutrum 3:** LAST SESSION None

Settings Menu:

- SimRum
- Akutrum 1
- Akutrum 3

Simcapture, when it is used and there are live sessions/simulation in the room” SimRum”

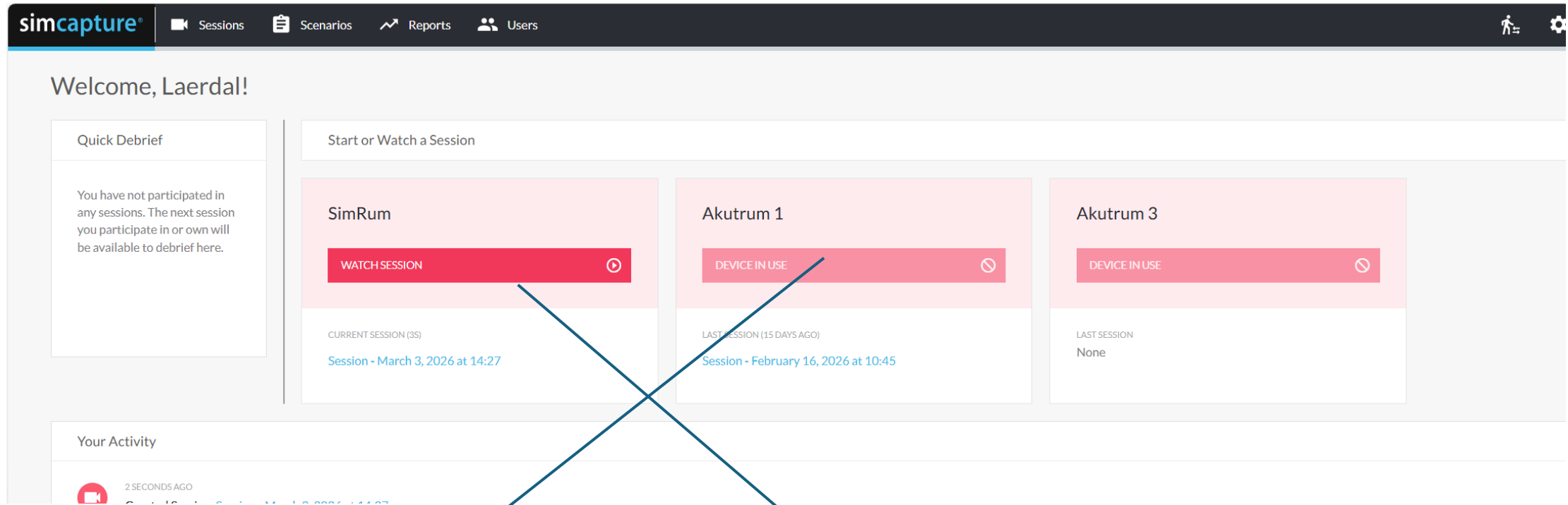

Possible with the right “userrole” and “organisation”, **become dark red**, to have liveview (watch sessions)

Session become **blured red**, admin don´t have access to live view, because they don´t have the needed role and orgnaisation

Simcapture node (computer) i simuleringrummet




In the controlrum

The Simcapture node
handle all recording in all 3
rooms






Start up on the controlpanel, choose “starta simrum” and it will look like the one to the right. On the panel you can select how you want to communicate

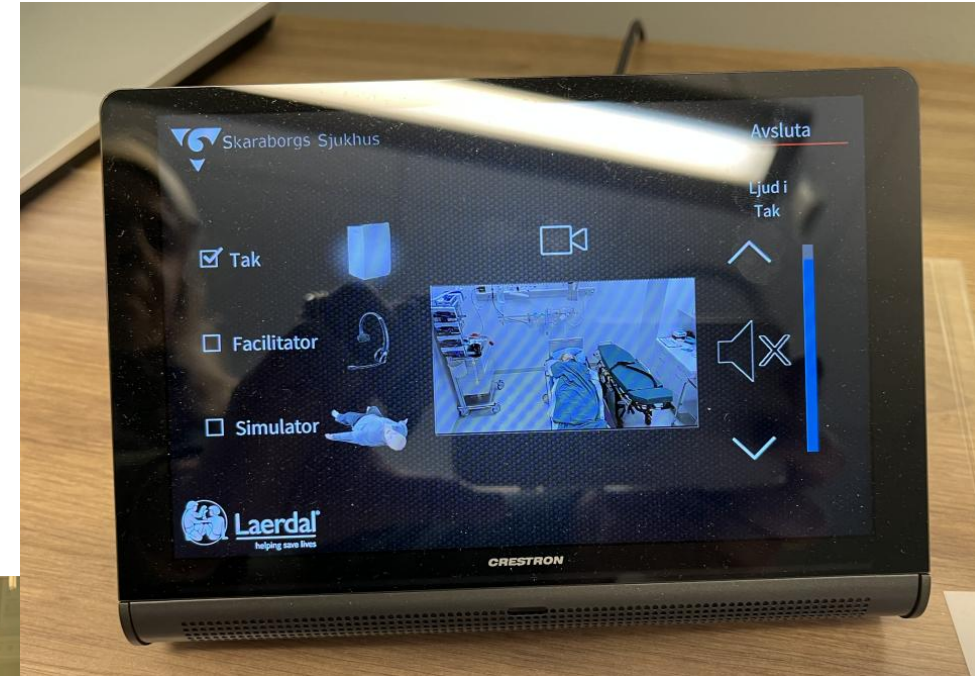

2 Headsets connected to the SC set up, remember to charge them.
1 ear : facilitator in the simulation room and 2 ears: persons in the control room

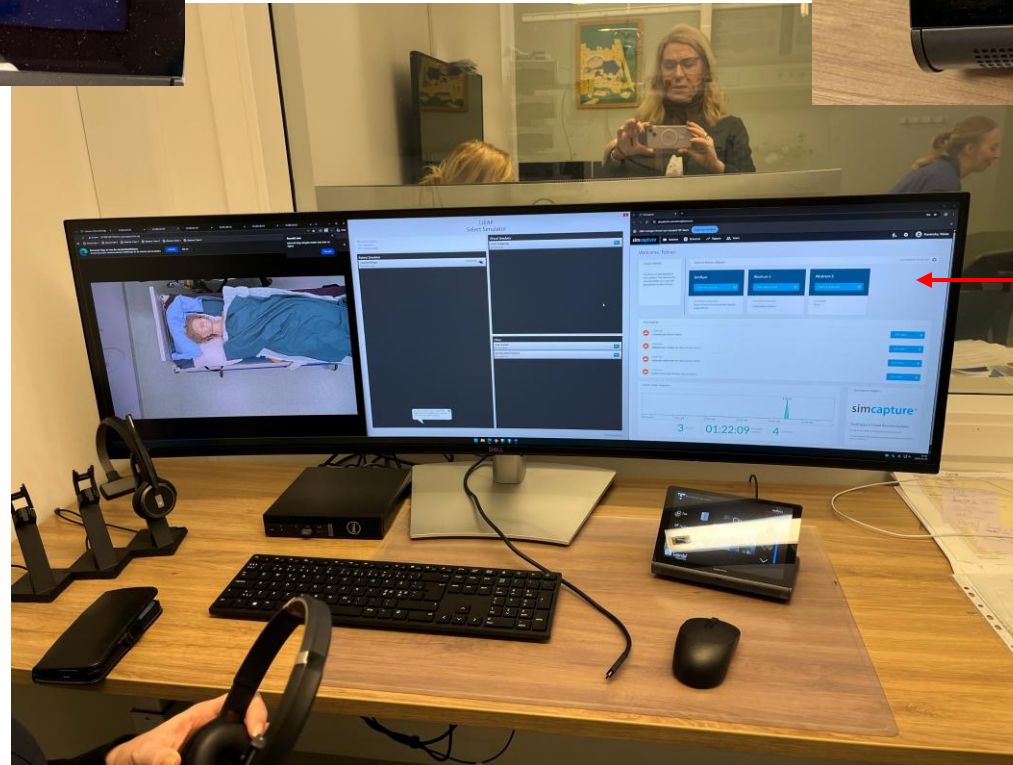

Be patient and wait until all programs are open and look like the big screen with 3 open programs – to the left: the window to the simulationrum, in the middle: LLEAP and to the right : Simcapture

Simulation room – rooms also used for real patients



Cameras and microphones are mandatory closed and need to be open



Click at the touch panel in the simulation room to start the cameras and the sound



Turning on the camera and microphone will be visitable and active in a session. Need to be close when the simulation is finish

Usertyper in Simcapture (few headlines and much more are included) *

System Admin


- Har full behörighet
- Kan se allt
- Kan göra nya passwords
- Kan lägga in användare
- Kan lägga in organisation
- Kan skapa annoteringar
- Kan skapa kurser
- Kan bestämma hur sessioner raderas
- Kan starta sessioner
- Kan se loggar för användare
- Har tillgång till statistik
- Har tillgång till System setup
- Kan skapa länk till alla sessioner att dela

Admin

- Har behörighet till sina egna sessions och kan se dem
- Kan skapa egna kurser
- Kan lägga in scenarios
- Kan bestämma hur egna sessions raderas
- Kan skapa länk till egna sessions att dela
- Kan starta sessioner

Användare/Participant

- Kan se egna sessioner
- Kan lägga in annoteringar i tilldelade sessioner

- And a lots of other functions connected to each functions
 - System admin have access to system set up via 

Create users (only System Admin can do this)

Users (10)

FULL NAME ^	ROLE	ORGANIZATION	LOGGED IN
<input type="checkbox"/> Brauer, Felicia	System Admin	Skarabores sjukhus Skövde Akuten	None

+ Add User More ▾

Users
Add New User Cancel Save

USER DETAILS

First name * Middle name Last name *

Username * Email * Password *
Passwords must be at least one character long.

User number
A user number could be an institution ID or other identifier.

Role ▾

- System Admin
- Administrator
- Participant

Cancel Save

→ Password can be changes by the users when they have log in



Organisation (system admin set up)

A user can have 1 or many organisations connected to a user

Dived the organisation into smaller group ex. “ Ambulans”, “barn akut”, medicinsk akut” etc.


Recomandation is not use “Skaraborgs sjukhus Skövde Akuten” but give users the smaller organisations they are working with

Create new organisations “ Click at Edit

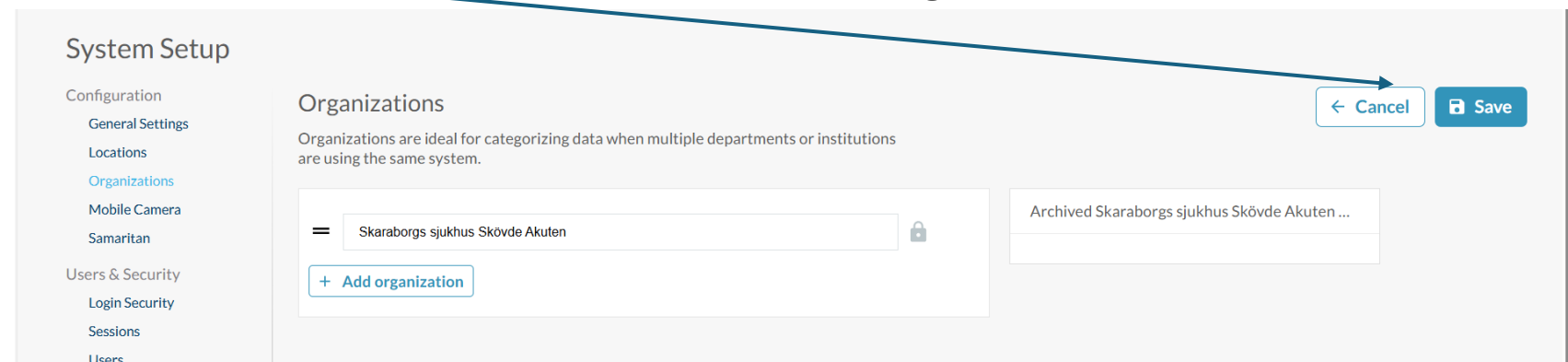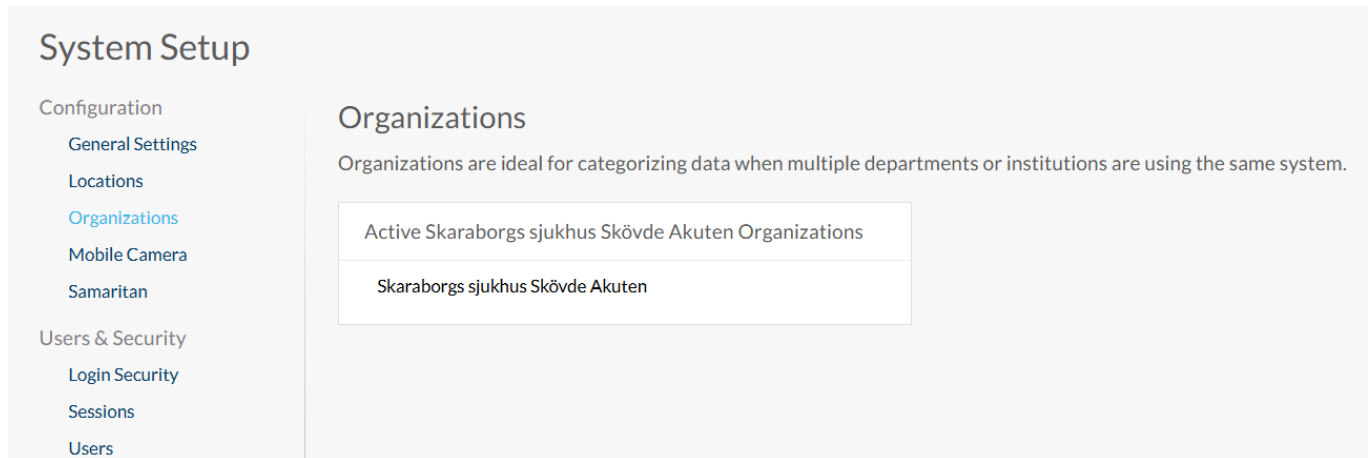

Create mobile camera and connect to location (only System Admin)

The screenshot shows the SimCapture System Setup interface. The top navigation bar includes 'simcapture', 'Sessions', 'Scenarios', 'Reports', and 'Users'. On the right, there are icons for a user profile, settings, and a user name 'SimCapture Support, Lae'. The main content area is titled 'System Setup' and has a left sidebar with a 'Configuration' section containing 'General Settings', 'Locations', 'Organizations', 'Mobile Camera', and 'Samaritan'. The 'Mobile Camera' section is active, showing a title 'Mobile Camera' and a subtitle 'You may pair up to 6 Mobile Camera devices and 2 Patient Monitor Capture feeds'. There are two buttons: 'Download PMC App' and 'Pair Device'. Below this is a table titled 'Paired Mobile Camera Devices' with columns for NAME, LOCATION, DEVICE ID, CREATED, and UPDATED. A blue arrow points from the 'LOCATION' column to the 'Locations' section in the sidebar of the second screenshot.

NAME ^	LOCATION	DEVICE ID	CREATED	UPDATED
 MalinsiPhone		31475a04-7df7-4956-9ca4-a806dd78a55b	Feb 26, 2026, 19:43	Feb 26, 2026, 19:44

The screenshot shows the SimCapture System Setup interface with the 'Locations' section active. The top navigation bar is the same as the first screenshot. The left sidebar has 'Locations' selected under the 'Configuration' section. The main content area is titled 'Locations' and has a subtitle 'Administrators can manage rec for rooms with multiple bays or'. Below this is a section titled 'Active Locations' with three input fields containing the text 'IVA', 'DT-lab 2', and 'Saneringen'.

Samaritan AI –transcription

simcapture®

Sessions Scenarios Reports Users



System Setup

- Configuration
 - General Settings
 - Locations
 - Organizations
 - Mobile Camera
 - Samaritan**
- Users & Security
 - Login Security
 - Sessions
 - Users

Samaritan configuration

[What is Samaritan?](#)

Use this page to enable or disable Samaritan for specific features, ensuring AI support aligns with your needs.

Feature

Session video transcription
Generate transcripts for newly captured session video

Status

✗ OFF

[Configure](#)

Will be launched with Swedish transcription in the next couple of days – choose “Swedish” as primary language

System Setup

Configuration

- General Settings
- Locations
- Organizations
- Question Categories
- Mobile Camera
- Learning Management System
- Samaritan**

Users & Security

- Login Security
- Sessions
- Users
- Courses
- Calendar
- Center Sign-In

Recording & Sessions

- Annotation Library

Session video transcription

Session video transcription On

How it works

Transcription language

Primary

English (US) (System language)

Additional languages

Samaritan will detect and transcribe Sessions in up to 4 additional languages

Please select a language

+ Add another language

Primary

- English (US) (System language)
- Chinese (Cantonese)
- Chinese (Mandarin, Simplified)
- Chinese (Mandarin, Traditional)
- Croatian
- Czech
- Danish
- Dutch
- English (Australia)
- English (India)
- English (Ireland)
- English (New Zealand)
- English (Scotland)

Off

Login security (only access for System Admin)

System Setup

Configuration

General Settings

Locations

Organizations

Mobile Camera

Samaritan

Users & Security

Login Security

Sessions

Users

Scenarios

Center Sign-In

Login Security

Administrators can manage general settings here

Logging In	
SESSION TIMEOUT AFTER INACTIVITY	TIME BEFORE IDLE WARNING
24 hours	10 seconds
LOCKOUT TIME AFTER MAX ATTEMPTS	INCORRECT ATTEMPTS ALLOWED BEFORE LOCKOUT
No requirement	Unlimited

Can be set by the System admin and can fit into the organisations roles. Click “edit” and adjust the setting

Password Policy	
PASSWORD STRENGTH	PASSWORD EXPIRATION
Weak	Never

Session premissions (only access for System Admin)

The screenshot displays the 'System Setup' interface. On the left is a navigation sidebar with categories: Configuration, Users & Security, Recording & Sessions, and About. The 'Sessions' option is highlighted in blue. The main content area is titled 'Sessions' and contains two panels. The 'Permissions' panel shows a single entry: 'Start, Edit, and Import Sessions Using Any Scenario Administrator'. The 'Sharing Defaults' panel lists three permission groups, each with three checked options: 'Session Contributor permissions default to' (View Results, Edit Session, Annotate Session), 'Session Participant permissions default to' (Annotate Session, View Others' Annotations), and 'Session Viewer permissions default to' (View All Annotations). A final section, 'Automatically Give Session View Permission to', has one checked option: 'Session Organization'.

System Setup

- Configuration
 - General Settings
 - Locations
 - Organizations
 - Mobile Camera
 - Samaritan
- Users & Security
 - Login Security
 - Sessions**
 - Users
 - Scenarios
 - Center Sign-In
- Recording & Sessions
 - Annotation Library
 - Recording Devices
 - Simulators
 - Applications
 - IP Video Feeds
 - IP Audio Feeds
- About

Sessions

Permissions

- Start, Edit, and Import Sessions Using Any Scenario Administrator**

Sharing Defaults

Session Contributor permissions default to

- View Results
- Edit Session
- Annotate Session

Session Participant permissions default to

- Annotate Session
- View Others' Annotations

Session Viewer permissions default to

- View All Annotations

Automatically Give Session View Permission to

- Session Organization

Annotation Library (created by System Admin)

System Setup

Configuration

- General Settings
- Locations
- Organizations
- Mobile Camera
- Samaritan

Users & Security

- Login Security
- Sessions
- Users
- Scenarios
- Center Sign-In

Recording & Sessions

- Annotation Library
- Recording Devices
- Simulators
- Applications
- IP Video Feeds
- IP Audio Feeds

About

- Version and Support

Support

- Internal Support
- App Features
- Single Sign-On






Annotation Library

Create your annotation library and annotation groups for use in sessions

← Cancel Save



+ Add Annotation Group

^ ABCDE (5) ↓

- =  A
- =  B
- =  C
- =  D
- =  E

+ Add Annotation

^ Kommunikation (3) ↓

- =  Presentation
- =  Identitetskontroll

System admin are the only one who can create and change annotations in Annotation library


Annotations can be selected when you create scenarios later on

Annotations can be created into a group or as uncategorized

Colour, sign and text can be changed

Users premissions (only access for System Admin)

(Possible to make a specific person to be the one) who can create and update users)

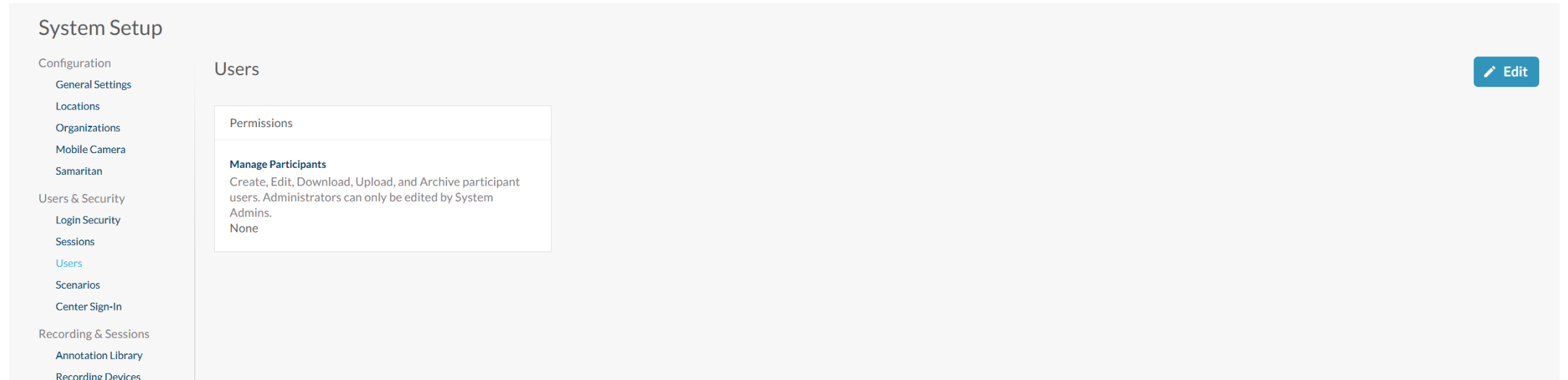

The screenshot shows a web interface for 'System Setup'. On the left is a navigation menu with categories: Configuration, Users & Security, and Recording & Sessions. Under 'Configuration', 'Users' is highlighted. The main content area is titled 'Users' and contains a 'Permissions' section with a sub-section 'Manage Participants'. The 'Manage Participants' section lists permissions: 'Create, Edit, Download, Upload, and Archive participant users. Administrators can only be edited by System Admins.' and 'None'. An 'Edit' button is located in the top right corner of the main content area.

System Setup

Configuration

- General Settings
- Locations
- Organizations
- Mobile Camera
- Samaritan

Users & Security

- Login Security
- Sessions
- Users**
- Scenarios
- Center Sign-In

Recording & Sessions

- Annotation Library
- Recording Devices

Users

Permissions

Manage Participants
Create, Edit, Download, Upload, and Archive participant users. Administrators can only be edited by System Admins.
None

Edit

Scenario permissions (only access for System Admin)

The screenshot displays the 'System Setup' interface. On the left is a navigation sidebar with categories: Configuration, Users & Security, and Recording & Sessions. The 'Scenarios' option is highlighted in blue. The main content area is titled 'Scenarios' and contains two panels. The 'Permissions' panel shows 'Create Scenarios' and 'Administrator' as the only user with permissions. The 'Sharing Defaults' panel shows four checked options for Scenario Contributor permissions: View Results, Debrief Sessions, Start, Join, and Edit Sessions, and Edit Scenario. There is also an unchecked option for 'Automatically Give Scenario View Permission to Scenario Organization'.

System Setup

Configuration

- General Settings
- Locations
- Organizations
- Mobile Camera
- Samaritan

Users & Security

- Login Security
- Sessions
- Users
- Scenarios**
- Center Sign-In

Recording & Sessions

- Annotation Library
- Recording Devices
- Simulators
- Applications
- IP Video Feeds

Scenarios

Permissions

- Create Scenarios
- Administrator

Sharing Defaults

Scenario Contributor permissions default to

- View Results
- Debrief Sessions
- Start, Join, and Edit Sessions
- Edit Scenario

Automatically Give Scenario View Permission to

- Scenario Organization

Restart - Recording devices (only System admin)

System Setup

Configuration

- General Settings
- Locations
- Organizations
- Mobile Camera
- Samaritan

Users & Security

- Login Security
- Sessions
- Users
- Scenarios
- Center Sign-In

Recording & Sessions

- Annotation Library
- Recording Devices

Recording Devices

Administrators can manage recording devices here

Node 1	Actions
Last status update March 3, 2026 at 14:06	Edit Reorder Restart

LOCATIONS
SimRum, Akutrum 1, Akutrum 3

HOST NAMES
DESKTOP-LSNPK8,10.189.129.136

CAPTURENODE ID
1cbbd8f0-ed91-5a98-86ce-3c0206d3772a [Copy](#)

MAC ADDRESSES
48210B519CFF

Node 1

Last status update March 3, 2026

LOCATIONS
SimRum, Akutrum 1, Akutrum 3

HOST NAMES
DESKTOP-LSNPK8,10.189.129

CAPTURENODE ID
1cbbd8f0-ed91-5a98-86ce-3c0206c

MAC ADDRESSES
48210B519CFF

VERSION
1.38.1+6de07bf

VIDEO/MONITOR INPUTS

- Axis Camera
- Axis Camera
- Axis Camera
- Axis Camera
- Axis Camera
- Axis Camera
- IP Encoder
- IP Encoder
- IP Encoder

ERRORS
None

[Export to Excel](#) [Register](#)

Green: Ready and connected
Grey: Not connected – try to restart
Yellow: off-line and need to be restarted

Create scenario (both System admin and admin)

Scenarios (2)

TITLE ^	OWNER	ORGANIZATION	UPDATED
<input type="checkbox"/> Demo: Strukturerat omhändertagande enligt ABCDE - Copy	Lind, Malin	Skaraborgs sjukhus Skövde Akuten	Mar 2, 2026, 16:19

1

[+ Add Scenario](#) [Import](#)

Search Scenario titles

2

simcapture Sessions Scenarios Reports Users

Demo: Strukturerat omhändertagande enligt ABCDE - Copy

Summary

OVERVIEW

Du jobbar på UVA. Det är morgon och du har precis påbörjat ditt pass. Av nattsjuksköterskan har du fått rapport om Jarl, en 80 år gammal man som har varit sjuk en längre tid. Han har haft svårt att behålla maten och aptiten har inte varit bra. Igår fick han akut ont i magen och uppsökte sjukhuset. Där konstaterades en inflammerad gallblåsa med viss påverkan på levern och han akutopererades inlett. Han fick sedvanlig vätskebehandling och det var inte någon stor blödning under operationen. Under natten har han haft smärtgenombrott och fått Oxynorm inj vid ett flertal tillfällen.

LEARNING OBJECTIVES

1. Utföra ABCDE och genomföra relevanta åtgärder
2. Arbeta i team
3. Kommunicera effektivt

Annotations

- Presentation
- Identitetskontroll
- Close Loop
- Tvätta händerna
- A
- B
- C
- D
- E

Settings

PUBLIC TITLE
Strukturerat omhändertagande

VIDEO DELETION POLICY
None

ORGANIZATION
Skaraborgs sjukhus Skövde Akuten

Attachments

NEWS2.webp

People

OWNER
Lind, Malin

CONTRIBUTORS
This Scenario has no contributors

VIEWERS
This Scenario has no viewers

Recent Activity

Title updated by Lind, Malin 2 DAYS AGO

3

2

Create new scenario and add title, overview, learnings objectives , add annotation

3

Add contributor(who can always work in and with the scenario and viewer (always live view)

Create scenario (both System Admin and Admin)

Settings

PUBLIC TITLE
Folke missar posten

VIDEO DELETION POLICY
5 Days

SIMULATOR
None

ORGANIZATION
Medicinsimulering

Attachments

[Add Attachments](#)

Sessions

This Scenario has not been used to start any sessions.

Create **public title** and the participants won't see what the training is about. EX: 82 years lady is confused and is found laying on the street

Video Deleting policy : The video, and only the video, will be deleted after the number of days after the recording

Add **simulators** in general settings and is able to be set as prework directly in the scenarios

Organisations : EX: “ medicinsimulering”, is the organisation who can live view in/from the session

Attachment, possible to share needed documentation but only as a read on document and only for admin and system admin

Start sessions from homepage (ex Akutrum 1)

When you start a session from a simulation rum it will turn dark red or blurred red. The colour deepens of the permission you must have for live view .

Dark red = permission to loiveview (and is based on the organisation you are connected to)

The screenshot shows the Simcapture user interface. At the top, there is a navigation bar with the Simcapture logo and menu items for Sessions, Scenarios, Reports, and Users. Below the navigation bar, a welcome message reads "Welcome, Laerdal!". The main content area is divided into several sections. On the left, there is a "Quick Debrief" section with a message: "You have not participated in any sessions. The next session you participate in or own will be available to debrief here." The central part of the page features a "Start or Watch a Session" section with three cards. The first card, "SimRum", has a light red background and a "DEVICE IN USE" button with a lock icon. Below it, it shows "LAST SESSION (6 DAYS AGO)" with the text "Demo: Strukturerat omhändertagande enligt ABCDE". The second card, "Akutrum 1", has a dark red background and a "WATCH SESSION" button with a play icon. Below it, it shows "CURRENT SESSION (15)" with the text "Folke tappar ansiktet". The third card, "Akutrum 3", has a light red background and a "DEVICE IN USE" button with a lock icon. Below it, it shows "LAST SESSION" with the text "None". At the bottom left, there is a "Your Activity" section.


Start up sessions –objection to choose before and during af session

The image displays three sequential screenshots of a software interface used for session preparation. A blue line connects the bottom of the first two screenshots to the text 'Preperation before start sessions'.

- Summary:** Shows session details including Scenario, Title, Organization (Skaraborgs sjukhus Skövde Akuten), Location (SimRum), Simulator (None), Date (February 26, 2026), Start Time (20:02), End Time (20:03), Notes (None), and Views (15).
- People:** Lists session participants and roles: OWNER (Lind, Malin), SCENARIO OWNER AND CONTRIBUTORS (SimCapture Support, Laerdal), CONTRIBUTORS (Edit to add/remove users), PARTICIPANTS (Lind, Malin3), and VIEWERS (Edit to add/remove users). It also includes an EXTERNAL LINK field.
- Annotate:** Provides a search bar and a list of annotation options: Presentation, Identitetskontroll, Close Loop, Tvätta händerna, and a list of letters A through E.

Preperation before start sessions


During and after start sessions –functions to be choosed before and during a session

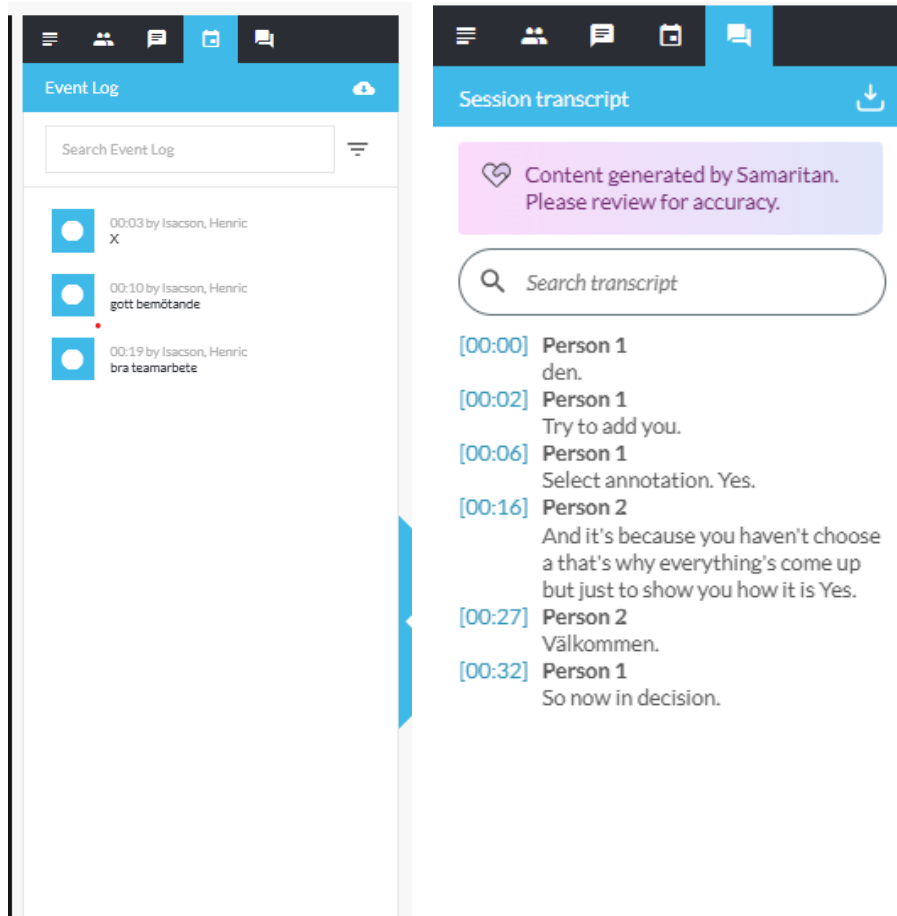

- Transcription will be in Swedish and you can search at specific words in the session
- Possible to search on word and click on the sence and find the exact part of the video connected to this words
- Many persons can be involved as person 1, 2, 3 etc.

Information, first logfile and the transcription*, after recorded sessions

Adjust camera focus and -zoom to prepare for the session and during the session

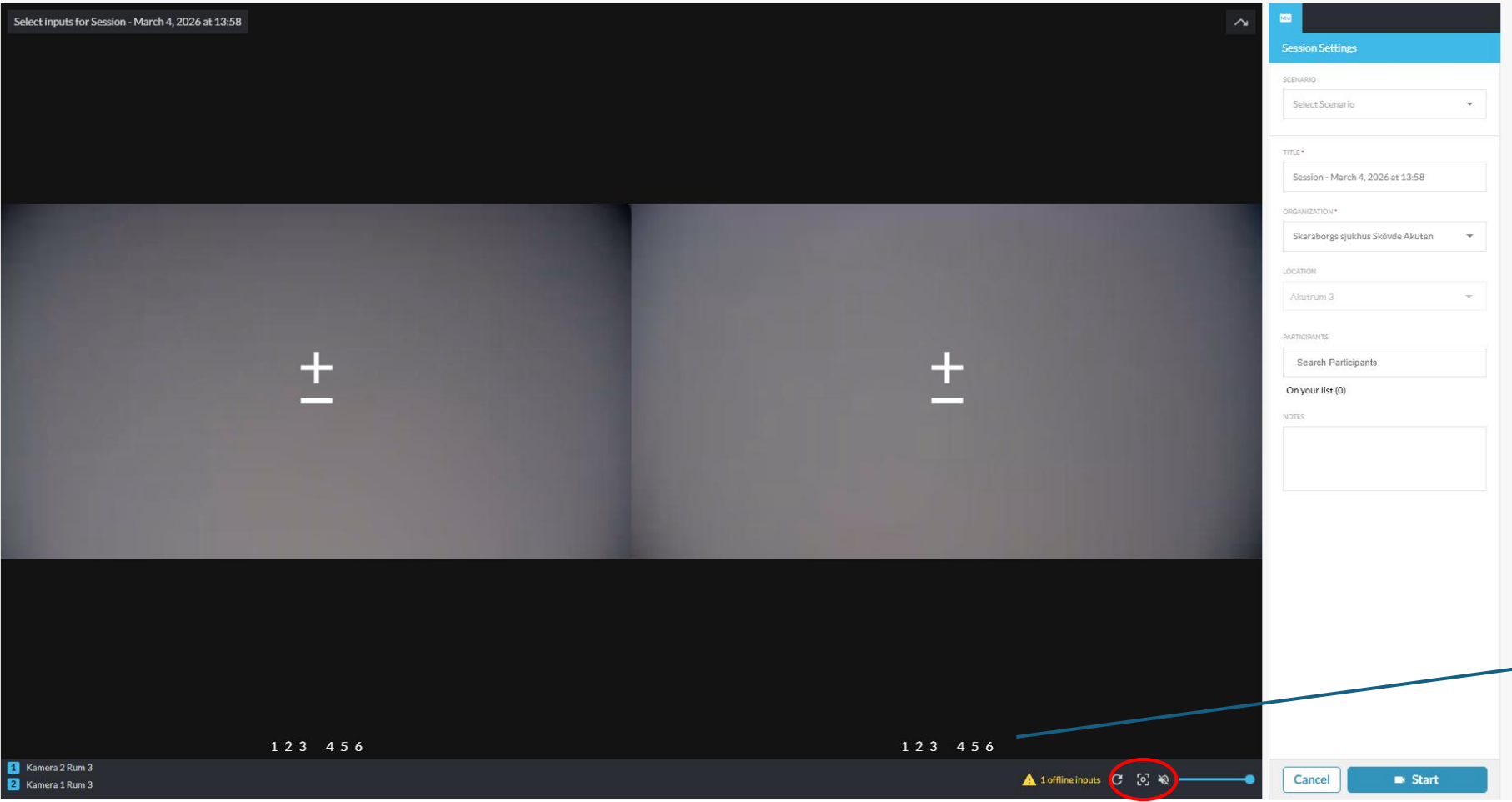

Each camera, if it is not at fixed camera, can zoom, change position etc. before and during a session




Possible to pre-set from position 1-6. Ask Laerdal for help.

Click on this to activate the zoom and setting

SimCapture Mobile Camera App



Preparation for SimCapture Mobile Camera

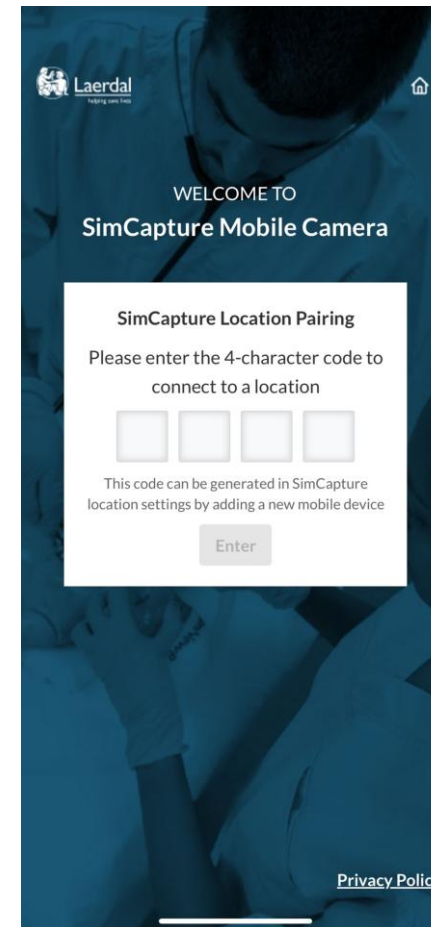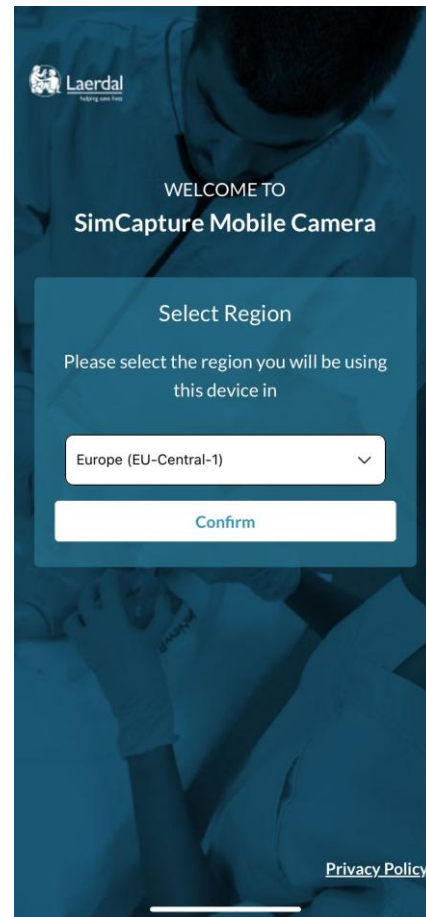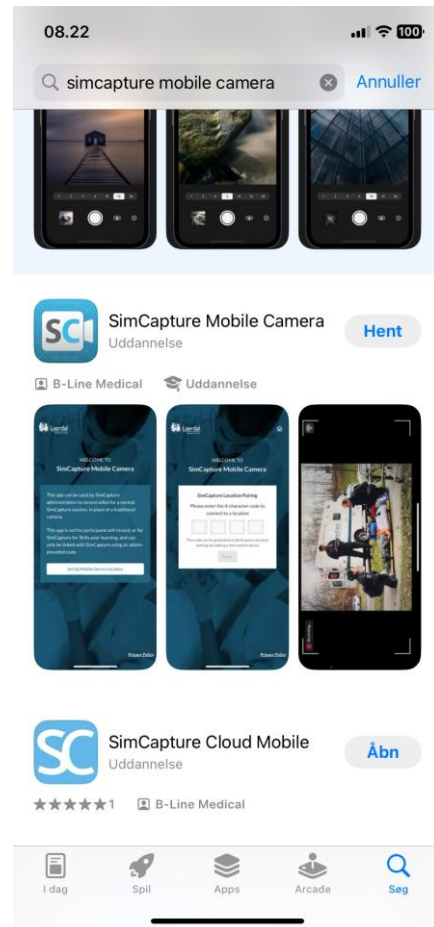

Installer Simcapture mobile camera

simcapture®

Sessions Courses Reports Users Calendar Control



System Setup

Configuration

General Settings

Inventory

Locations

Organizations

Question Categories

Mobile Camera

Users & Security

Login Security

Sessions

Users

Courses

Calendar

Center Sign-In

Recording & Sessions

Annotation Library

Recording Devices

Simulators

System setup all settings, or learn more

Configuration

Configuration and date preferences for your center location

General Settings

Date and time, Language, Site Name, Notifications, and URL

Inventory

Inventory Item categories and scenarios and

Locations

Add Locations to place

Organizations

Associate Users with different

Question Categories

Organize questions

Mobile Camera

Pair mobile devices for recording

Mobile Camera

You may pair up to 6 Mobile Camera devices and 2 Patient Monitor Capture feeds

Paired Mobile Camera Devices

NAME ^	LOCATION	DEVICE ID	CREATED	UPDATED
Felicia	Saneringen	f61f424c-3a25-45bd-9818-3306325370e2	Mar 4, 2026, 14:33	Mar 4, 2026, 14:34
Herman		484166af-741d-4501-9a6e-ff296ecb87f8	Mar 4, 2026, 14:32	Mar 4, 2026, 14:33
MalinsiPhone		31475a04-7df7-4956-9ca4-a806dd78a55b	Feb 26, 2026, 19:43	Feb 26, 2026, 19:44

Download PMC App

Pair Device

Patient Monitor Capture is a companion desktop app that can be used to screen-record a patient monitor tool as one of your session video inputs.

Recording & Sessions

Recording Settings center around Configure default settings for devices

Annotation Library Add

Recording Devices Set

Simulators Add

Applications Add

Video Feeds Add

Audio Feeds Add

simcapture® Sessions Scenarios Reports Users

Home, Laerdal!

< Debrief

Users who have not participated in sessions. The next session participants will participate in or own will be available to debrief here.

Start or Watch a Session

Saneringen

START NEW SESSION

LAST SESSION
None

Mobile Camera

SimRum

START NEW SESSION

LAST SESSION (6 DAYS AGO)

Demo: Strukturerat omhändertagande enligt ABCDE

Install Simcapture patient monitor capture(PMC app)

Access the Simcapture browser at <https://uia-pro.simcapture.com/> on the device that will be used for the patient monitor and click on “download PMC app”.

Open the app on, for example, the patient monitor and select “Europe” and then click on “pair devices” to get a 4-digit code for the patient monitor. It is then named and connected to an in-situ location.

There are different icons on devices depending on whether it is a camera or patient monitor (see example).

Skövde Emergency has a total of 4 mobile cameras and 2 patient monitors available. 2 additional mobile cameras will arrive shortly.

NOTE - It is easy to “pair” and “unpair”, so you can change how you use the extra rooms

Now with a mobile in- situ location

The screenshot displays the SimCapture dashboard interface. At the top, a dark navigation bar contains the 'simcapture' logo and menu items for Sessions, Scenarios, Reports, and Users. On the right side of the navigation bar, there are icons for a person, a gear, and a user profile labeled 'SimCapture Support, Laerdal'. Below the navigation bar, the main content area is titled 'Welcome, Laerdal!'. On the left, a 'Quick Debrief' section contains a message: 'You have not participated in any sessions. The next session you participate in or own will be available to debrief here.' The central part of the dashboard is titled 'Start or Watch a Session' and features a 'Last Updated: in 0 seconds' indicator with a refresh icon. This section contains four session cards: 'Saneringen', 'SimRum', 'Akutrum 1', and 'Akutrum 3'. Each card has a 'START NEW SESSION' button with a plus icon. The 'Saneringen' card is highlighted with a black circle around its 'Mobile Camera' icon. Below each card, the 'LAST SESSION' status is shown: 'None' for Saneringen and Akutrum 3, and specific session details for SimRum and Akutrum 1. At the bottom, a 'Your Activity' section is partially visible.

simcapture

Sessions Scenarios Reports Users

SimCapture Support, Laerdal

Welcome, Laerdal!

Quick Debrief

You have not participated in any sessions. The next session you participate in or own will be available to debrief here.

Start or Watch a Session

Last Updated: in 0 seconds

Saneringen

Mobile Camera

START NEW SESSION

LAST SESSION: None

SimRum

START NEW SESSION

LAST SESSION (6 DAYS AGO): Demo: Strukturerat omhändertagande enligt ABCDE

Akutrum 1

START NEW SESSION

LAST SESSION (38 MINUTES AGO): Folke tappar ansiktet


Akutrum 3

START NEW SESSION

LAST SESSION: None


Your Activity

Start simulering med Simcapture Mobile Camera

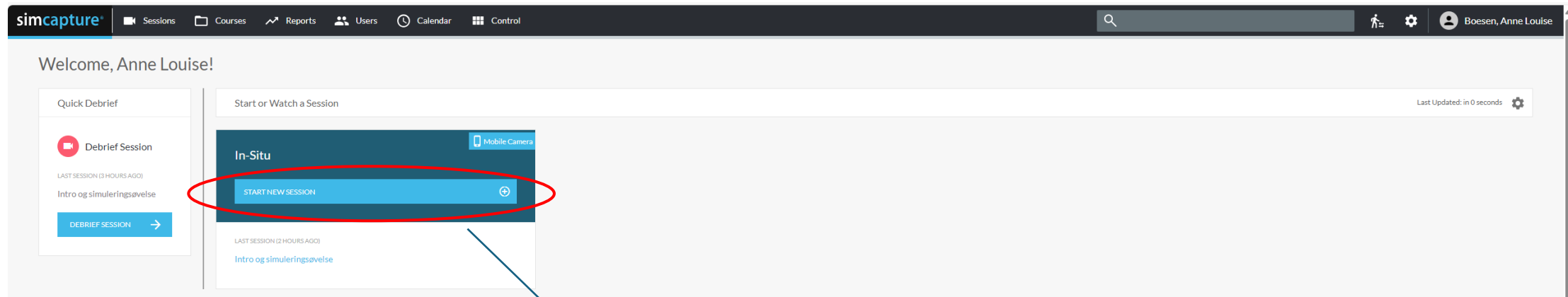


Prepare the mobile cameras and optionally place them on a tripod to get a recording that is not too shaky.

Press "Connect to in-situ", then the cameras are ready and nothing more needs to be done with them.

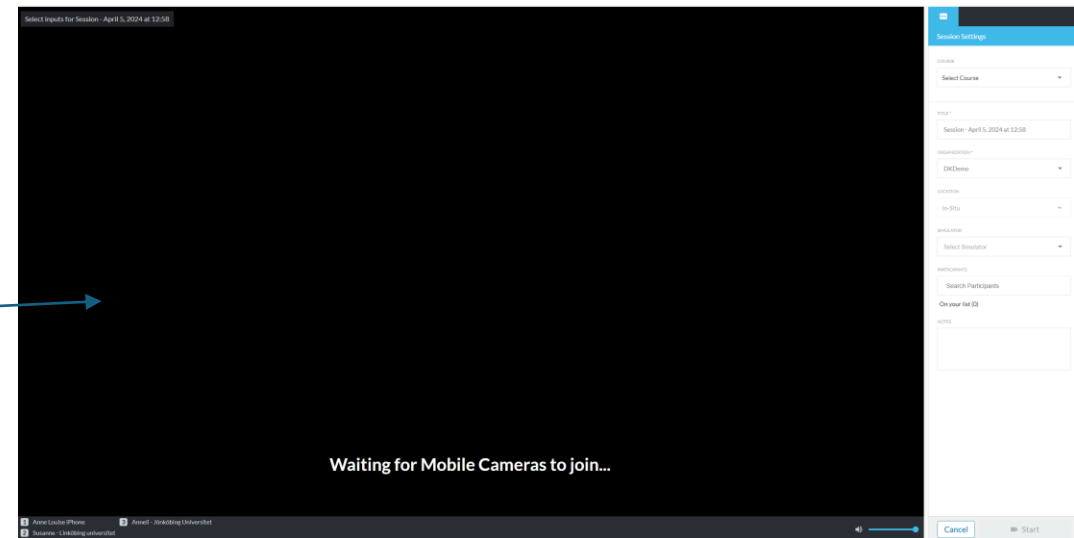

Lyd – ved brug af mobile kameraer.

The sound, when recording with mobile cameras, will always come from 1 video feed and it will always be at the top left side. It is possible to move the images around so that the audio video feed comes from another camera. Important: Remember that the sound from 3-4 devices cannot be combined, but the sound will always reflect what has been recorded via the camera that is at the top left.

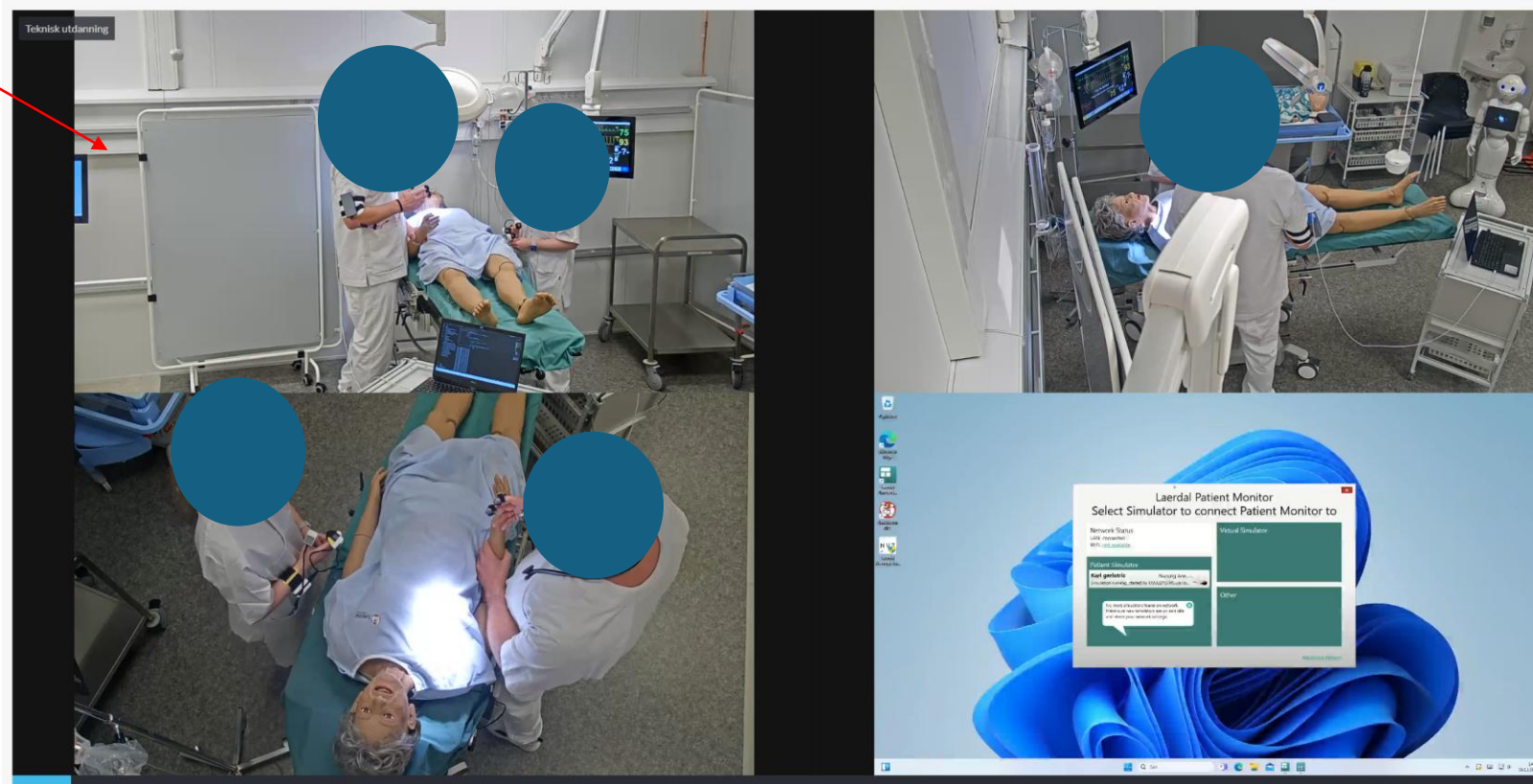

Create in – situ location

imcapture | [Icons: Camera, Folder, Line Graph, People, Clock, Grid, Search] | [Icons: User, Gear, Profile] Laerdal, SimCapture Support

System Setup

Configuration

- General Settings
- Inventory
- Locations**
- Organizations
- Question Categories
- Mobile Camera

Users & Security

- Login Security
- Sessions
- Users
- Courses
- Calendar
- Center Sign-In

Recording & Sessions

- Annotation Library
- Recording Devices
- Simulators
- Applications

System setup allows for the configuration of various aspects of the SimCapture system. Select one of the options to the left to access the corresponding settings, or learn more about what configuration options are available below.



Configuration

Configuration Settings involve basic system setup and information. Set time and date preferences, configure your email and notification settings, and setup your center locations here.


General Settings	Date and time, Language, Site Name, Notifications, and URL
Inventory	Inventory Items are supplies associated with scenarios and sessions.
Locations	Add Locations where sessions will take place
Organizations	Associate Users, Scenarios, and Sessions with different Organizations
Question Categories	Organize questions into categories
Mobile Camera	Pair mobile devices for use with Session recording

Users & Security

Simcapture mobile off-line

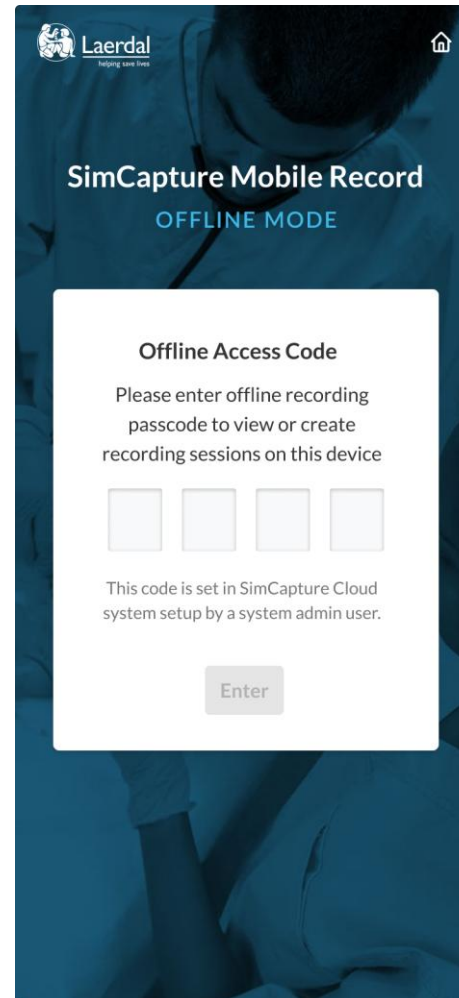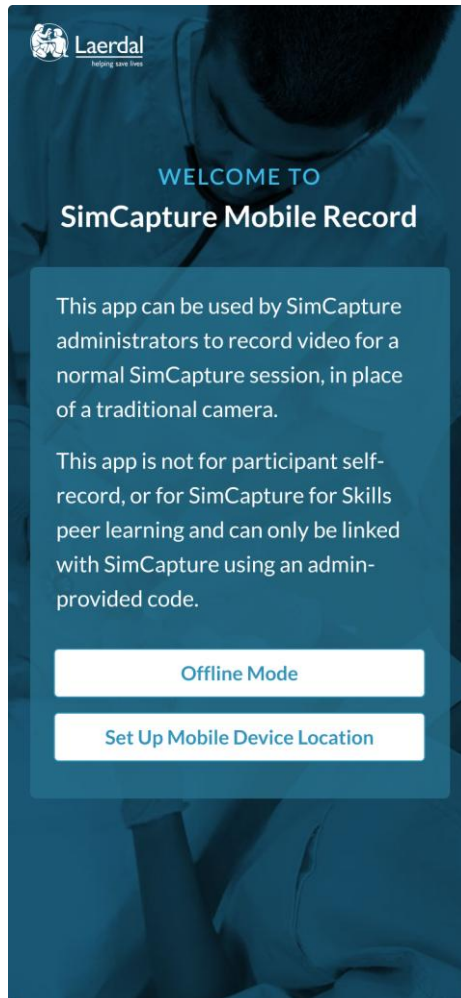

Passcode for accessing Offline Recording is created by the user in SimCapture Cloud. This is a one-time set-up task to secure the app feature.

 **Laerdal**
helping save lives

SimCapture Mobile Record

OFFLINE MODE

[Record New Session](#)

Offline Activities

Session title 5 Uploaded

Location: Sim Station 23
Recorded: 07/24/24 at 02:14 pm
Duration: 15m43s

[Upload](#) [Playback](#) [Delete](#)

Session title 4 Uploading...

Location: Sim Station 23
Recorded: 07/24/24 at 02:14 pm
Duration: 15m43s


[Upload](#) [Playback](#) [Delete](#)


Session title 3 Not uploaded

Location: Sim Station 23
Recorded: 07/24/24 at 02:14 pm
Duration: 15m43s

[Upload](#) [Playback](#) [Delete](#)



 **Laerdal**
helping save lives



SimCapture Mobile Record


OFFLINE MODE


New Session Details


Session title *

Institution

Location

Course
 

Scenario
 

Session Owner *
 

[Start Recording](#)




Feature in Simcapture Mobile kamera off- line

- Record from *one device at a time*, for future upload




- *Before recording is started, switch from front or back-facing camera*
- Annotate *offline sessions* from the mobile device

Debriefing

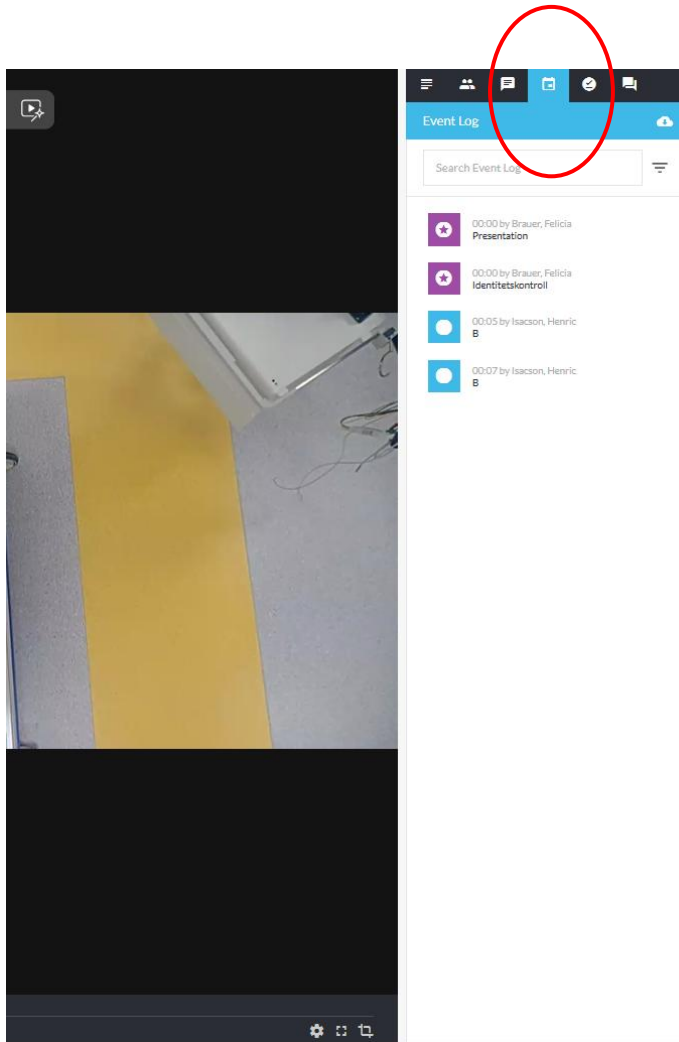


Log file with comments and annotation.
Mark directly on the annotation log and video will move to the place

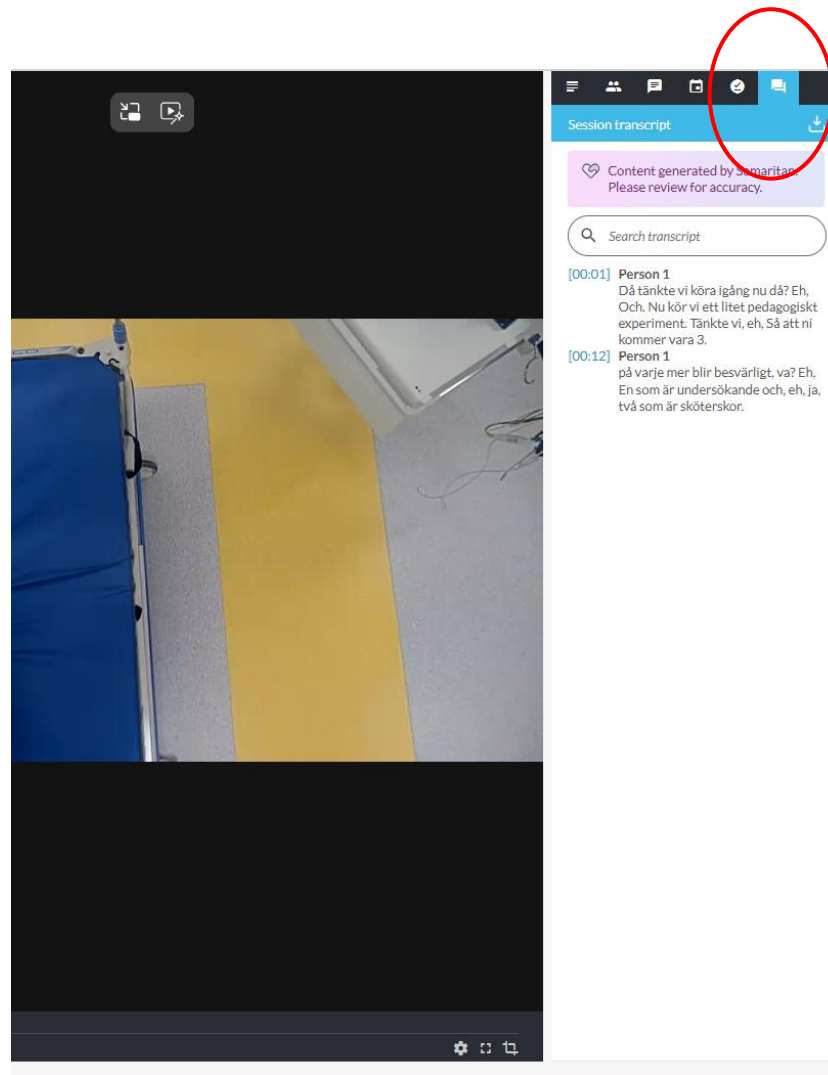

Transcription from video session, can be used in debriefing
Mark directly on the transcription log and video will stop on the time in the video as you want

Recommendation:
Always keep the video stopped,
choose the annotation or
transcription spot and start the
video again.


Debriefing

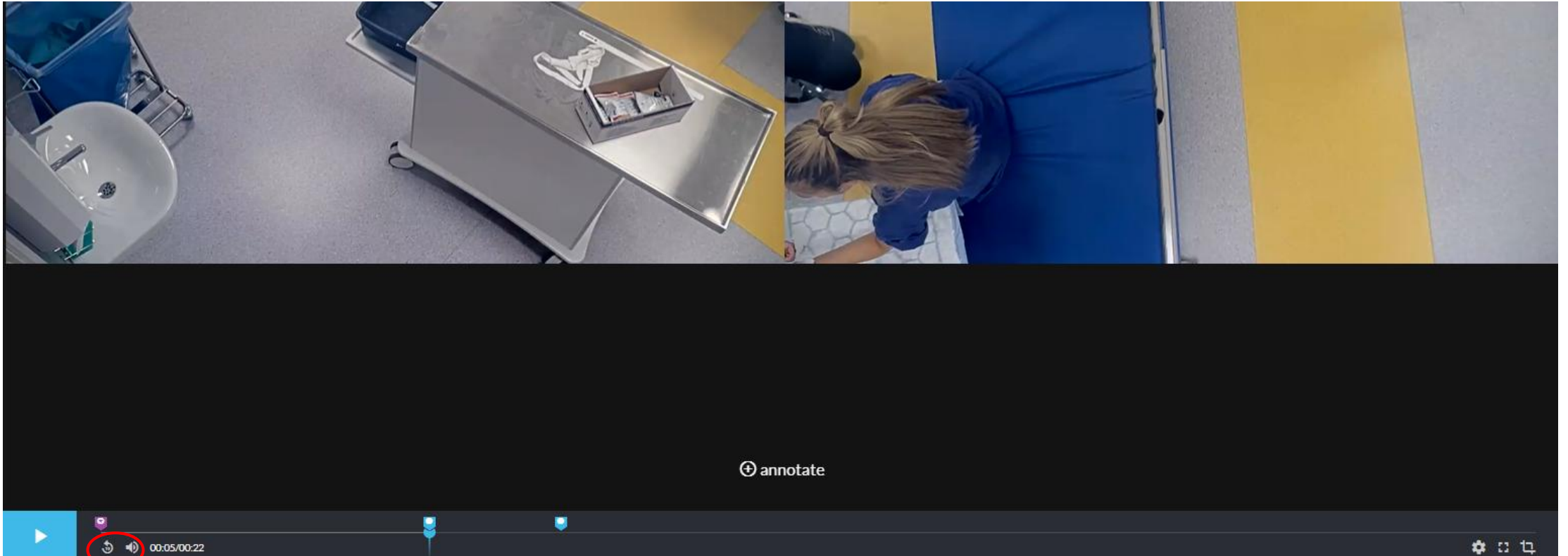

When you have marked the place in the video as you want, click in the icon and you will go 10 sec back, if you want to go 20 sec. back you need to click twice

Recommendation:
Always keep the video stopped, choose the annotation or transcription spot and start the video again.

Import af videoer/film

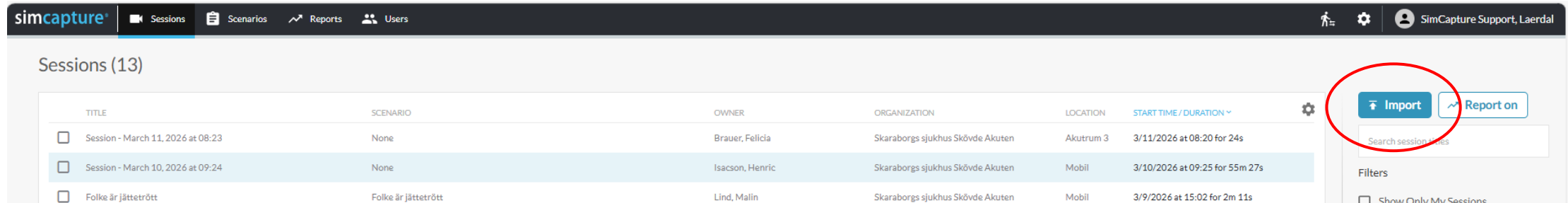

The screenshot shows the SimCapture web interface. The top navigation bar includes 'simcapture', 'Sessions', 'Scenarios', 'Reports', and 'Users'. The user profile is 'SimCapture Support, Laerdal'. The main content area is titled 'Sessions (13)' and contains a table with the following data:

TITLE	SCENARIO	OWNER	ORGANIZATION	LOCATION	START TIME / DURATION
<input type="checkbox"/> Session - March 11, 2026 at 08:23	None	Brauer, Felicia	Skaraborgs sjukhus Skövde Akuten	Akutrum 3	3/11/2026 at 08:20 for 24s
<input type="checkbox"/> Session - March 10, 2026 at 09:24	None	Isacson, Henric	Skaraborgs sjukhus Skövde Akuten	Mobil	3/10/2026 at 09:25 for 55m 27s
<input type="checkbox"/> Folke är jättetrött	Folke är jättetrött	Lind, Malin	Skaraborgs sjukhus Skövde Akuten	Mobil	3/9/2026 at 15:02 for 2m 11s


On the right side of the table, there is a search bar with the text 'Search session titles' and two buttons: 'Import' (circled in red) and 'Report on'. Below the search bar, there is a 'Filters' section with a checkbox for 'Show Only My Sessions'.

Possible to add all kind of downloaded videos from YouTube, other sources etc.
The video can be used as normal, and you can debrief and make annotation as you want.

Raderings policy


Standard “deleting policy” – 2 Manuele options

1) Possible to delete manually on sessionslevel

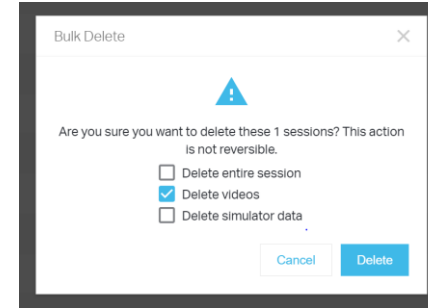

2) System admin and Admin can add video deleting policy to a scenario and this will work everytime the scenario is used

EX: 1 = 1 day, ¹

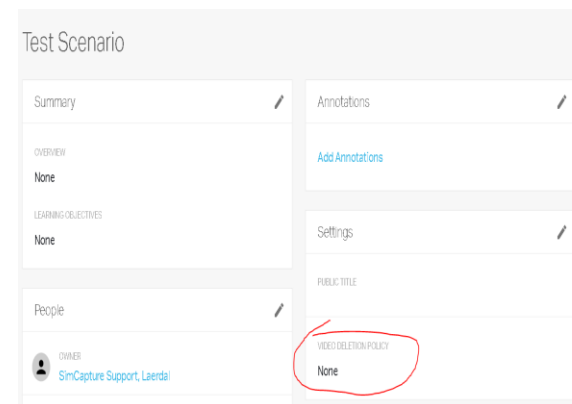

Deleting policy – 3. options
- need to be installed by laerdal


3) Global Auto Deleting:

All sessions are automatically deleted after a set number of days, which is a minimum of 1 day or several days.

+/- Possibility to, at the scenario level, save videos longer

- Delete rule via Scenario – when is the video deleted

Video deleting policy, ex =1 (1 dag)

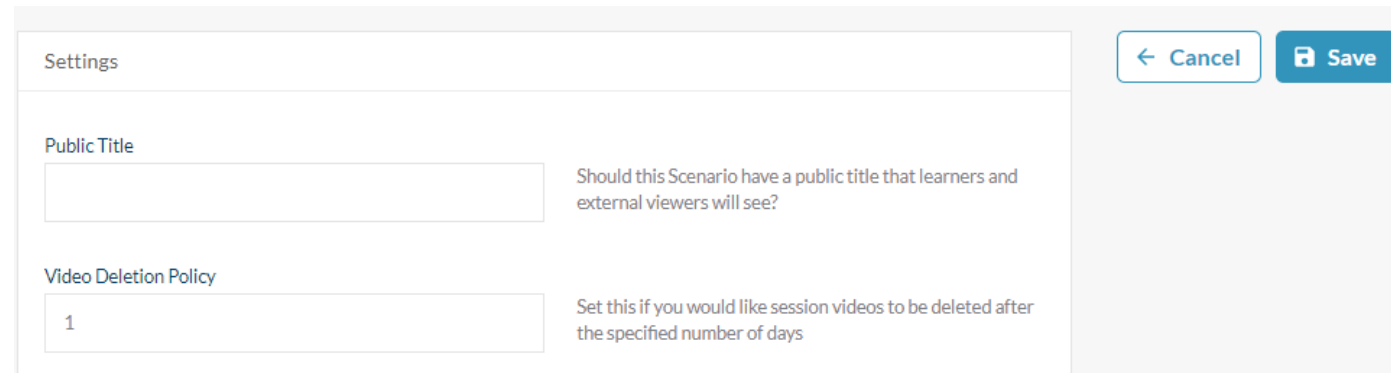

The screenshot shows a settings panel with two main sections. The first section is 'Public Title' with an empty text input field and a description: 'Should this Scenario have a public title that learners and external viewers will see?'. The second section is 'Video Deletion Policy' with a text input field containing the number '1' and a description: 'Set this if you would like session videos to be deleted after the specified number of days'. To the right of the settings panel are two buttons: a 'Cancel' button with a left-pointing arrow and a 'Save' button with a floppy disk icon.


Every day, at 12:00 noon, an update runs and then the videos are deleted if the delete function has been used (e.g., 1 day) via the scenario.

Ex. If a recording is made at 8:00 on Monday, the video will be deleted at 12:00 on Tuesday.

Ex. If a video is recorded at 1:00 PM on Monday, it will be deleted on Wednesday at 12:00 PM, i.e., 47 hours later.

N.B. Videos can be at least 25 hours old and at most 47 hours old.

Prevent deleting (sessions may not be deleted)

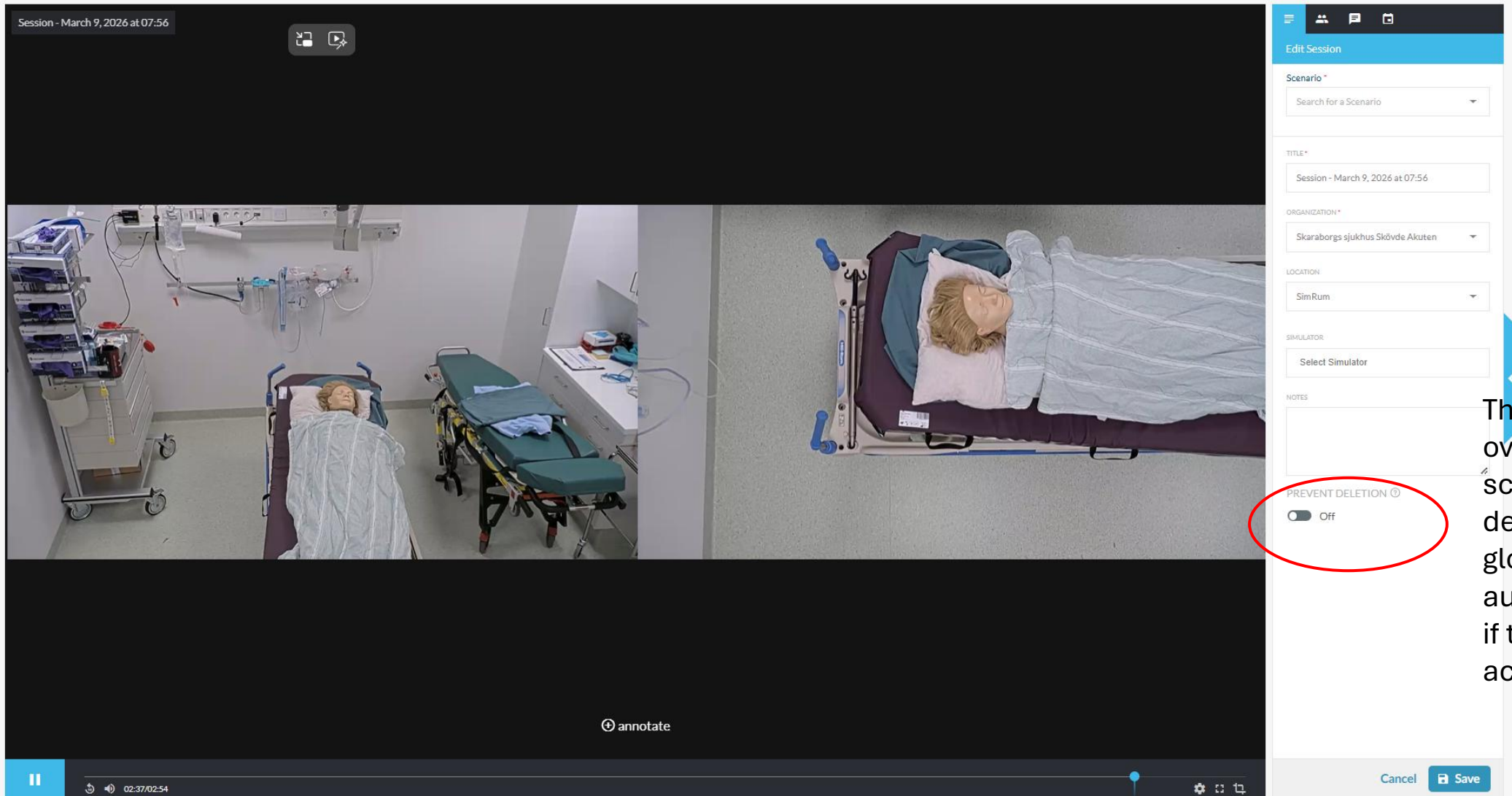

The screenshot displays a simulation environment. The main view is split into two panels: the left panel shows a hospital room with a patient on a gurney, and the right panel shows a top-down view of the patient on the gurney. On the right side, there is an 'Edit Session' panel with various settings. A red circle highlights the 'PREVENT DELETION' toggle, which is currently set to 'Off'. Below the toggle, there are 'Cancel' and 'Save' buttons.

Session - March 9, 2026 at 07:56

Edit Session

Scenario *

Search for a Scenario

TITLE *

Session - March 9, 2026 at 07:56

ORGANIZATION *

Skaraborgs sjukhus Skövde Akuten

LOCATION

SimRum

SIMULATOR

Select Simulator

NOTES

PREVENT DELETION ©

Off

annotate

Cancel Save

This function overrule scenario deleting and global autodeleting (if this is activated)

Swedish helpdesk

Arbetstimmar:

Helgfria vardagar:8-16

Vid problem så önskar vi att ni först och främst ringer eller mailar våran helpdesk:

helpdesk.sverige@laerdal.com

Telefonnumret är detsamma som till kundservice: +46 85 56 14 610 men väljer ni knappval 3 så kommer ni till helpdesk. Både kundservice och helpdesk finns på plats helgfria vardagar kl. 8-16.